

TABLE OF CONTENTS

1. Indications for using The Elator
2. Brief description of The Elator
3. Putting on The Elator
4. Delayed diagnosis of other conditions
5. Alternative treatments
6. Warnings
7. Care of
8. Product warranty



1. INDICATIONS FOR USING THE ELATOR


The Elator can be used by men with erectile dysfunction (ED) or without ED as a tool to make intercourse possible in spite of having no erection or a partial or weak erection, and to increase the duration or frequency of intercourse. The Elator is suggested when medication, injections, surgery or other therapies are not advised or have been unsuccessful in treating ED.

2. BRIEF DESCRIPTION OF THE ELATOR

The Elator is an external penile support device designed to comfortably and gently add just enough rigidity to the penis to make intercourse possible every time it is used, with or without any erection. The Elator allows for full skin sensation and pleasure for both partners during intercourse. To perform intercourse with The Elator no medication, injection or surgery is needed, therefore no side-effects can occur. You can use The Elator with or without a condom without reducing the safety or effectiveness of either.


The Elator components:

- A Soft Flexible Loop
- B Sliding Latch
- C Flexible Support Tubes
- D Base Lock
- E Base Ring

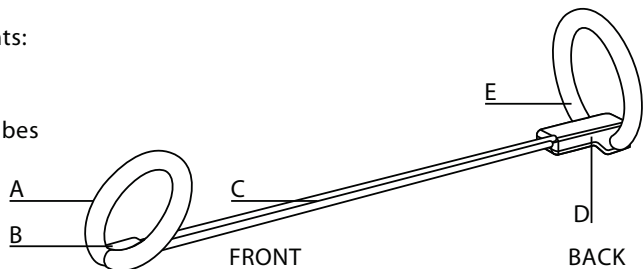


Be Careful!


The Elator is made to be flexible, but bending it too far or too often can cause damage!

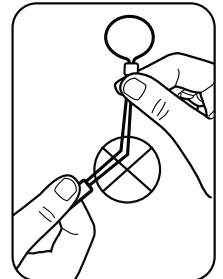


3. PUTTING THE ELATOR ON

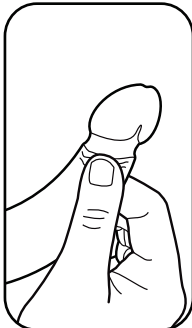


Fully retract the skin under the penis head to expose the penis shaft.

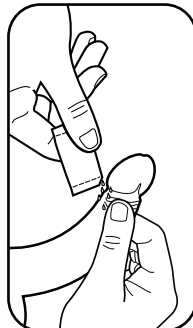


Apply a small amount of lubricant around the circumference of the exposed penis shaft.

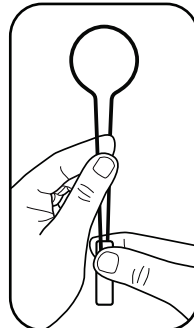


Move Sliding Latch down to the base of the Elator. Spread the Flexible Tubes to open the Flexible Loop.

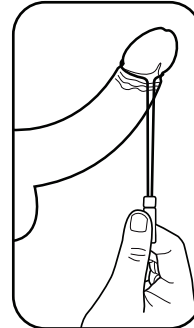


Insert the penis head under the Flexible Loop, centering the tubes and Sliding Latch directly under the penis head.

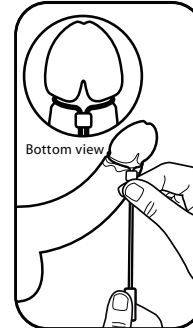

Move the Sliding Latch forward all the way, tightening the Flexible Loop. The Sliding Latch locks the Flexible Loop around the penis head.

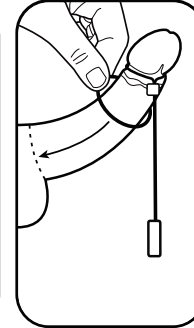

Place the Base Ring over the penis shaft bringing the ring down to the base of the penis.

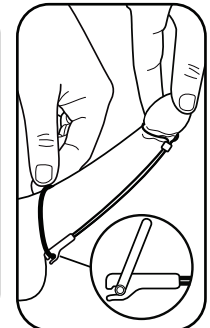

Stretch your penis and lock the Base Ring at the Base Lock. If you have a lot of skin around the penis head, make sure the skin is not pinched. You should feel very comfortable.

4. DELAYED DIAGNOSIS OF OTHER CONDITIONS

If you have symptoms of erectile dysfunction, i.e., inability to achieve an erection that is sufficient for sexual intercourse, you may consult your physician prior to using this device to avoid a potentially harmful delay in diagnosing any of the most common causes of this condition, such as diabetes, multiple sclerosis, cirrhosis of the liver, chronic renal failure, alcoholism or others.

5. ALTERNATIVE TREATMENTS


You can use alternative treatments for your ED such as: counseling, drug therapy, hormonal therapy, vascular surgery, and implanted prosthetic devices. You may contact your physician for alternative treatment options.

6. WARNINGS

1. To avoid potential tissue irritation or unpleasant sensations while using The Elator always apply personal lubricants on all surfaces of The Elator that contact the penis.
2. To make penetration easier at the beginning of intercourse, always prepare with adequate foreplay and use personal lubricants.
3. Use water based or silicone based personal lubricants. Reapply lubricants during intercourse as needed by either partner. Do not use heavy petroleum based products (e.g. Vaseline) for personal lubrication.
4. Duration of intercourse with The Elator should not exceed 30 minutes during any single sexual encounter. We strongly recommend a 60 minute rest interval between repeated sexual encounters with The Elator.
5. Stop using The Elator if you or your partner have unpleasant sensations during intercourse or have insufficient amount of natural or applied lubricants.
6. Do not use The Elator if either partner is under the influence of alcohol or legal or illegal drugs that may impair judgment or alter skin sensations or awareness, since such use may impair either partner's judgment and perception leading to increase risk of injury to the penis or vagina.
7. Do not use The Elator if you have decreased hand strength or dexterity because this might make safe and correct application or removal of the device impossible or difficult.
8. Do not use The Elator unless you are properly trained and able to take full responsibility for the safe use of this product, and have full knowledge how to correctly put on, use and take off The Elator.
9. Do not fall asleep while wearing The Elator since prolonged inactivity while wearing The Elator may cause permanent injury to the penis.
10. Do not use The Elator as a penis extender device. The Elator is designed and intended for intercourse only.
11. Use of The Elator does not prevent pregnancy, nor does it prevent or protect either partner from any sexually transmitted disease.
12. If either partner feels discomfort or unusual sensation while wearing The Elator, immediately discontinue use until the cause of the sensation is determined and corrected. Further, if any particular sexual position causes discomfort during intercourse, change position.

7. CARE OF THE ELATOR

After each use wash The Elator thoroughly under warm water using antiseptic soap. Allow the Elator to fully dry. Slide the Sliding Latch (B) completely forward to the front and then place the protective case. Each Elator is manufactured in the USA using only FDA-approved materials in an FDA-approved facility and carries a 12-month warranty.

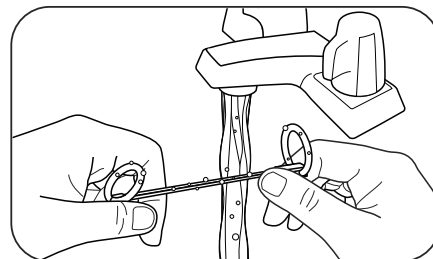

8. PRODUCT WARRANTY

The Elator will give a full refund for the product if the product has not been used or opened and returned within 60 days. If the Elator bag or seals have been tampered with in anyway, no refund will be given. Since full genitalia contact is implied and assumed in Elator external penile support device usage, disease transmission is an overriding and vital concern for all involved parties – buyer, user, manufacturer and vendor. If bag or seal has been tampered with, unit will be returned to customer and no refund will be issued.

If the Elator is found to be actually defective or display poor craftsmanship, please contact us for a RMA number via email sales@theelator.com for call us at 878-356-1066 within 1 year of original purchase date. Replacement procedures as described within the manufacturer's extended warranty period (see below).

The Elator External Penile Support Product Warranty

The Elator warrants, to the original purchaser only, that its products a) substantially conform to The Elators published specifications and b) are substantially free from defects in material and workmanship. This warranty expires one year from the date of product delivery. All warranty service will be performed at the company's facility at no charge to the purchaser. If Elator is unable to repair the product to conform to the warranty, Elator will provide at its option one of the following: a replacement product or a refund of the purchase price for the non-conforming product. If it is discovered that user negligence has occurred and product failure or damage is the result of improper use or care there will be no warranty issued in this case and the damaged product will be discarded or shipped back to the user at his cost.

THE FOREGOING WARRANTIES ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

In no event will Elator be liable for any special, incidental, or consequential damages based on breach of warranty, breach of contract, negligence, strict tort, or any other legal theory. Damages that Elator will not be responsible for include but are not limited to, loss of profits; loss of savings or revenue; loss of use of the product or any associated equipment; cost of capital; cost of any substitute equipment, facilities or services; downtime; claims of third parties including customers; and injury to property.

Technical Support

If a purchaser believes that a product is not operating in substantial conformance with Elators published specifications or there appear to be defects in material and workmanship, the purchaser should contact our technical support representatives. If the problem cannot be corrected over the telephone and the product and problem are covered by the warranty, the technical support representative will authorize the return of the product for service and provide shipping information. If the product is out of warranty, repair or replacement charges will be quoted. All non-warranty repairs receive a 90-day warranty.